



LiquiBand® Octyl 0.4

The high strength 2-Octyl-cyanoacrylate for safe and effective wound closure.




✓ 2-OCTYL CYANOACRYLATE FORMULA

- ▶ Durable and flexible coverage
- ▶ High viscosity
- ▶ Competitive wound closure strength when compared to market leading devices

✓ HIGH PATIENT SATISFACTION

- ▶ Low exothermic reaction
- ▶ Atraumatic wound closure

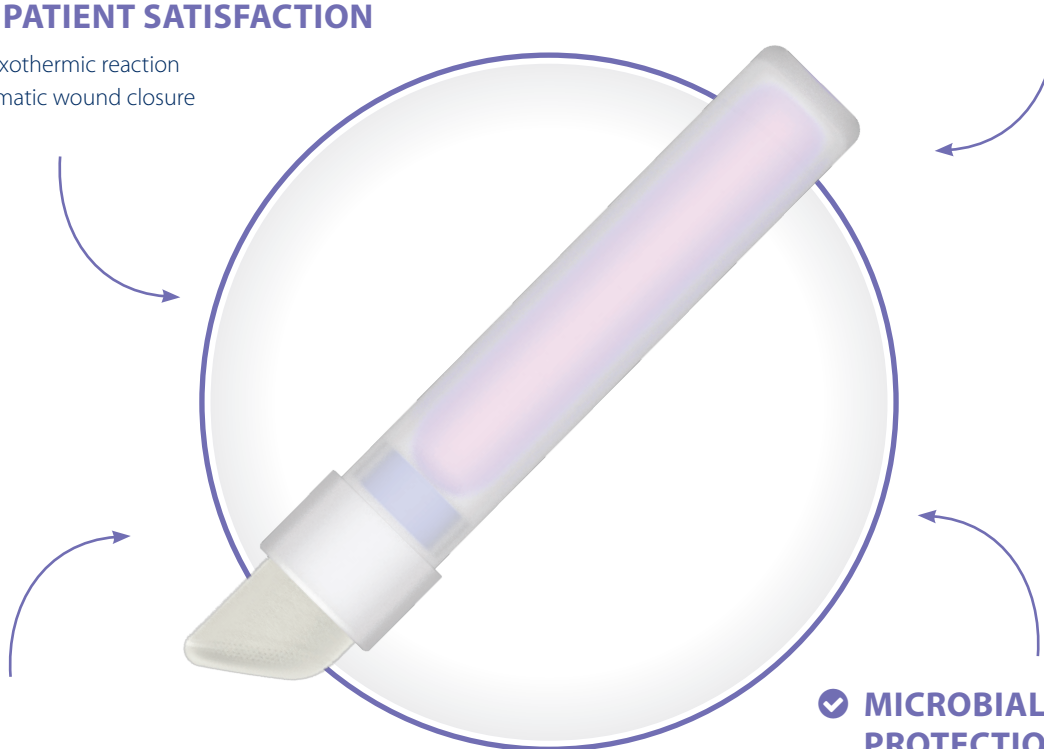

✓ EASY TO USE APPLICATOR


- ▶ Ready to use out of the box
- ▶ Contoured tip for controlled delivery of adhesive
- ▶ Transparent for visualization

✓ MICROBIAL BARRIER PROTECTION

- ▶ Effective barrier to gram positive, gram negative and fungal microbes


Application Guide *LiquiBand® Octyl 0.4*

- ▶ Ensure that wound is thoroughly clean and dry prior to closure.

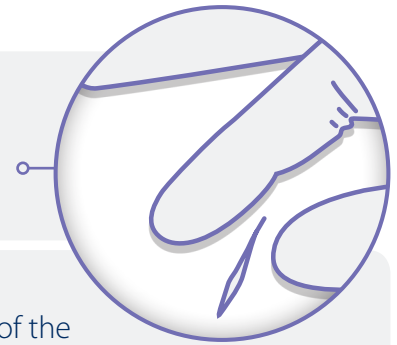

- ▶ While holding upright, squeeze middle of the LiquiBand® Octyl 0.4 device until a cracking sound is heard.

- ▶ Invert the tip and gently squeeze to prime the device. When the tip turns purple, LiquiBand® Octyl 0.4 is ready to use.

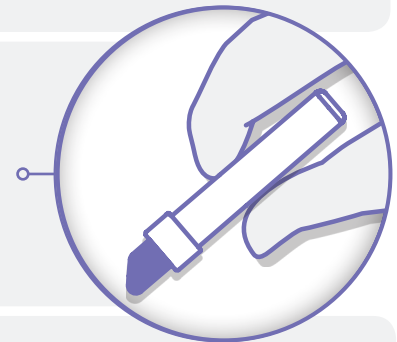

- ▶ While holding the wound edges together, apply a single, even coat of LiquiBand® Octyl 0.4 over the length of the wound, using the tip of the device to distribute and prevent pooling of the adhesive.
- ▶ Continue to hold the wound edges together for approximately 60 seconds.

LiquiBand® Octyl 0.4

Size	Qty Per Box	Product Code
0.4g	6	72014001